

## Liberty Drain Bag Inservice

### To Use:

1. Remove protective cap from catheter adapter and connect to catheter.
2. Position tubing to ensure an unimpeded flow down to the drainage bag. Hang drainage bag on bed frame.
3. Secure tubing to sheet with bed sheet clamp.




Reinforced hanger with built-in anti-kink drain tube guide



Anti-reflux provides protection to hinder back flow.

## Liberty Drain Bag Inservice

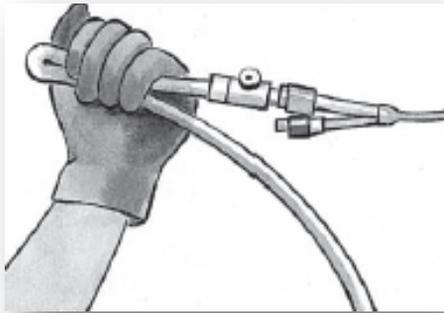

### Using the Sample Port:

1. Perform hand hygiene.
2. Don non-sterile gloves
3. Kink tubing a minimum of 3 inches below the sample port until urine is present under puncture site. Cleanse surface of puncture site. 3. Insert needle (sharp or blunt) through center of puncture site with alcohol.
4. Attach blunt plastic cannula syringe and aspirate desired volume of urine.
5. Transfer urine to specimen container.



## Liberty Drain Bag Inservice

### To Drain Bag:

1. Perform hand hygiene.
2. Don non-sterile gloves.
3. Remove slide tap from holder.
4. Open valve and empty into a collection container. Do not allow the tube to touch the container.
5. After emptying the bag, close valve and clean end of drain tube with alcohol before placing it in the holder.

